

Designing Compact, Efficient and High Performing Multiparameter Patient Monitors



Leni Skariah

Continuous measurement of patient vital signs like heart rate, respiration rate and blood oxygen saturation (SpO2) is crucial for effective care, and the ability to measure those simultaneously makes multiparameter patient monitors increasingly important.

Electronic patient monitors use noninvasive sensors attached to patients to collect and display physiological data. For multiparameter patient monitors, one of the major challenges is achieving a compact size and low power consumption while retaining high performance.

Today, patients that are confined to hospital beds are attached to bulky and expensive multiparameter patient monitors with wires and are not mobile, preventing widespread usage in mobile nursing and home healthcare environments. There is a need for compact multiparameter patient monitors that can be wirelessly attached to a patient and transmit data either wired or wirelessly to display on a mobile phone, tablet or computer.

TI's "Multiparameter front end reference design for vital signs patient monitor" is a compact, low-cost, low-power reference design that incorporates the necessary front-end components to collect key vital signs.

The design captures parameters like electrocardiogram (ECG), SpO2, heart rate and respiration using the AFE4403 and the ADS1292R biosensing front-end integrated circuits. The design interfaces to the [Software-configurable cardiac pacemaker detection module reference design](#) to enable pace-pulse detection and connects to three 0.1 Celsius accurate temperature sensors (TMP117) to measure skin temperature. To measure ECG, the design uses standard wet ECG electrodes, and it also uses a transmissive type finger clip sensor to measure SpO2. The captured raw data is available for transfer to a back end/processor or a PC over an isolated Universal Asynchronous Receiver Transmitter (UART) interface at 460.8 Kbps using an onboard MSP432P401 microcontroller. A standard USB port or a lithium-ion battery charged from the USB powers the design.

Figure 1 shows the reference design block diagram.

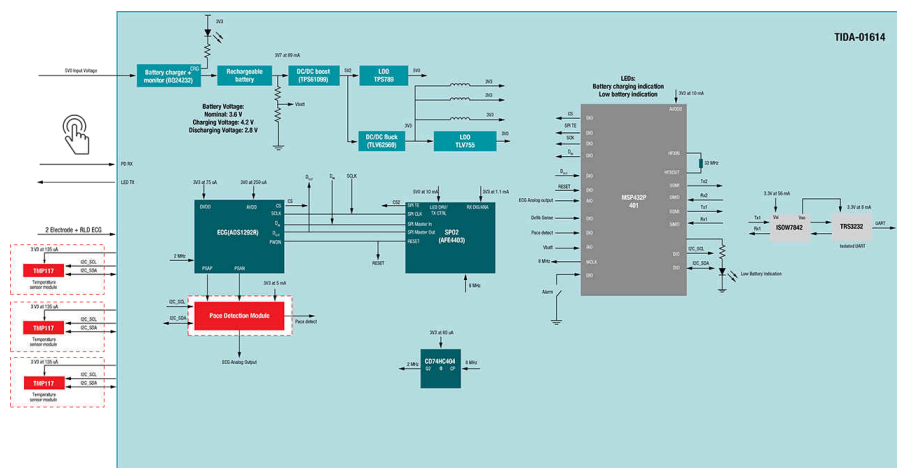


Figure 1. Multiparameter Front-end Subsystem for Vital Sign Patient Monitoring Reference Design

A graphical user interface (GUI) displays the captured waveform for a 5-second moving interval while basic filtering cleans up noise such as common-mode interference from power, lights, etc. [Figure 2](#) displays ECG

results, respiration and SpO2, along with heart rate in beats per minute, respiration number (breaths per minute), SpO2 percentage and temperature values on a PC.



Figure 2. GUI Displaying ECG, PPG, Respiration, Heart Rate and Skin Temperature

Figure 3 shows a measured ECG, photoplethysmography (PPG), pacemaker, heart rate and temperature sensor data on a computer. When a pace pulse is detected, a pace marker shows the presence of pace pulse as indicated by the top line in Figure 3.



Figure 3. GUI Displaying ECG, PPG, Pacemaker, Heart Rate and Skin Temperature

Conclusion

The reference design was created so that you can evaluate the devices working together. It comes with a design guide, schematics, layout and bill of materials to help quickly evaluate and accelerate product development. It facilitates real-time capture of vital sign parameters.

What are your biggest challenges in patient monitoring applications? Tell us in the comments.

Additional Resources

- Check out these blog posts:
 - “[Going beyond traditional SpO2 measurement with multiwavelength optical measurements.](#)”
 - “[No strings attached: Creating next-generation wireless patient monitors.](#)”
 - “[Optical heart-rate sensors for biometric wearables.](#)”
 - “[10 lessons learned from 50-plus biometric wearable product development cycles.](#)”
- Download the Analog Design Journal article, “[Improving battery life in wearable patient monitors and medical patches.](#)”

- Learn more about ultra-low power wired & wireless connectivity solutions across the SimpleLink MCU Platform: www.ti.com/simplelink.

IMPORTANT NOTICE AND DISCLAIMER

TI PROVIDES TECHNICAL AND RELIABILITY DATA (INCLUDING DATA SHEETS), DESIGN RESOURCES (INCLUDING REFERENCE DESIGNS), APPLICATION OR OTHER DESIGN ADVICE, WEB TOOLS, SAFETY INFORMATION, AND OTHER RESOURCES "AS IS" AND WITH ALL FAULTS, AND DISCLAIMS ALL WARRANTIES, EXPRESS AND IMPLIED, INCLUDING WITHOUT LIMITATION ANY IMPLIED WARRANTIES OF MERCHANTABILITY, FITNESS FOR A PARTICULAR PURPOSE OR NON-INFRINGEMENT OF THIRD PARTY INTELLECTUAL PROPERTY RIGHTS.

These resources are intended for skilled developers designing with TI products. You are solely responsible for (1) selecting the appropriate TI products for your application, (2) designing, validating and testing your application, and (3) ensuring your application meets applicable standards, and any other safety, security, regulatory or other requirements.

These resources are subject to change without notice. TI grants you permission to use these resources only for development of an application that uses the TI products described in the resource. Other reproduction and display of these resources is prohibited. No license is granted to any other TI intellectual property right or to any third party intellectual property right. TI disclaims responsibility for, and you will fully indemnify TI and its representatives against, any claims, damages, costs, losses, and liabilities arising out of your use of these resources.

TI's products are provided subject to [TI's Terms of Sale](#) or other applicable terms available either on [ti.com](#) or provided in conjunction with such TI products. TI's provision of these resources does not expand or otherwise alter TI's applicable warranties or warranty disclaimers for TI products.

TI objects to and rejects any additional or different terms you may have proposed.

Mailing Address: Texas Instruments, Post Office Box 655303, Dallas, Texas 75265
Copyright © 2023, Texas Instruments Incorporated